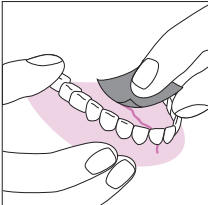
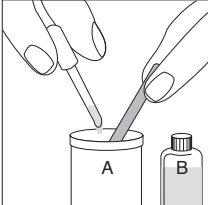
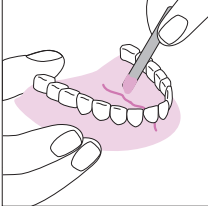
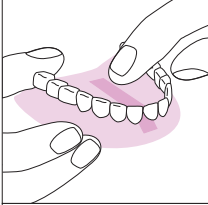
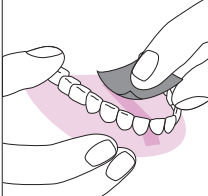
# E PROTESAN<sup>®</sup>

**RAPIDO**

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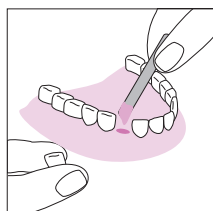
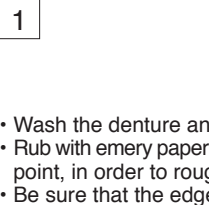
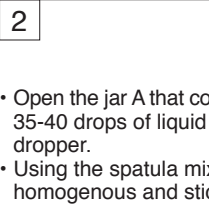
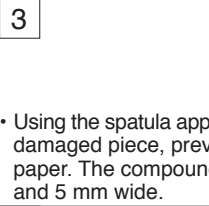
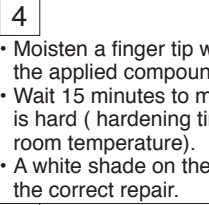
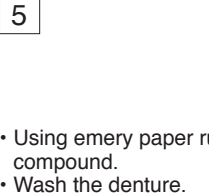
# GB INSTRUCTIONS FOR USE

PROTESAN RAPIDO is fit to repair damaged dentures (when they are not broken in pieces) and when a tooth in resin or porcelain comes unstuck from movable dentures made of resin, (or made with a metallic skeleton).

PROTESAN RAPIDO is made of two components: one liquid (methilmetacrylate) and one in powder (polymethilmetacrylate). When mixed together the two components make a resin, that hardens by itself. To get a satisfactory repair it is necessary to follow up in sequence all the instructions.

	1	<ul style="list-style-type: none"> <li>Wash the denture and dry it carefully.</li> <li>Rub with emery paper about 5mm near the damaged point, in order to roughen the surface.</li> <li>Be sure that the edges are closely fixed.</li> </ul>
	2	<ul style="list-style-type: none"> <li>Open the jar A that contains the dosed powder, add 35-40 drops of liquid from the jar B, using the dropper.</li> <li>Using the spatula mix the paste till it becomes a homogenous and sticky compound.</li> </ul>
	3	<ul style="list-style-type: none"> <li>Using the spatula apply the compound on the whole damaged piece, previously rubbed with emery paper. The compound has to be about 2 mm thick and 5 mm wide.</li> </ul>
	4	<ul style="list-style-type: none"> <li>Moisten a finger tip with liquid B and make even the applied compound.</li> <li>Wait 15 minutes to make sure that the compound is hard (hardening time can vary according to the room temperature).</li> <li>A white shade on the denture does not jeopardize the correct repair.</li> </ul>
	5	<ul style="list-style-type: none"> <li>Using emery paper rub off incidental residuals of compound.</li> <li>Wash the denture.</li> </ul>

## When a tooth comes unstuck from the denture

- With a cotton pad soaked with liquid B clean tooth and tooth socket.
- Open jar A (that contains the dosed powder) and using the dropper add about 35 drops of liquid from the jar B.
- Using the spatula mix the two components in order to obtain an homogeneous compound, not particularly thick.
- Insert a little amount of compound in the socket, than insert the tooth.
- Moisten a finger tip with liquid B, than remove the exceeding compound from around the tooth.
- Wait 15 minutes and wash the denture.

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## GB CAUTIONS

Components are inflammable. Keep Protesan Rapido out from heat and naked flames. Accurately avoid that the product contacts your eyes. In case of contact, wash it with plenty of hot water. Do not use the product in case of tested allergy to the components. Do not use the product on wounds or abrasions. Never use the product on the internal side of denture in contact to the gum. The consumer is responsible for the correct use.

Repair does not eliminate the grounds that caused the break, therefore, even if the product has a strong resistance, you are recommended to consult your dentist. Odontotechnic can make the final repair if the instructions are correctly followed.

KEEP PROTESAN RAPIDO OUT OF REACH OF CHILDREN.  
KEEP THE PRODUCT IN A COOL (< 25° C) AND AERATED PLACE FAR FROM HEAT.

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