I PONTEFIX®

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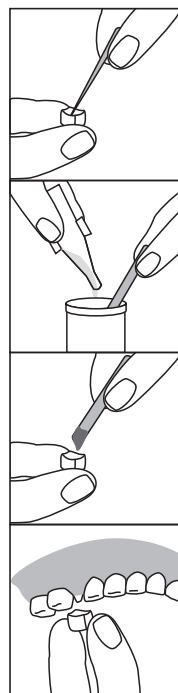
GB PONTEFIX®

INSTRUCTIONS FOR USE

PONTEFIX TEMPORARY REPAIRS BRIDGES, CROWNS, PINNED TEETH AND DENTAL CAPSULES, WHATEVER MATERIAL ARE MADE OF. TO MAKE AN EASIER COMPREHENSION IN THESE INSTRUCTIONS THE WORD "CAPSULE" IS USED TO INDICATE ANY OF THE ABOVE MENTIONED ELEMENTS. THE PECULIAR FORMULA OF PONTEFIX ENABLED TO USE ZINC OXYPHOSPHATE FOR TEMPORARY REPAIRS.

- PONTEFIX PRESERVES THE CHARACTERISTICS OF A DEFINITIVE FIXING.
- PONTEFIX IN ANY CASE DOES NOT DAMAGE YOUR CAPSULES.
- PONTEFIX DOES NOT IMPEDE THE FURTHER OPERATION OF YOUR DENTIST.
- THE CONTENT OF THIS PACKAGING MUST BE USED ONLY ONCE.

Before using please read carefully all the instructions

- Remove any food residuals from the capsule, by means of a sharp tool. Then complete the cleaning process with a cotton pad soaked in alcohol.
- Insert the cotton roll between gum and cheek, near the tooth or the pin you want to repair.
- Control the capsule in order to put the right side directly over tooth.

- Open the powder container and add **all the liquid** from the monodose container.
 - Mix with the spatula to obtain a homogeneous compound.
- WARNING**
The paste hardens in about two minutes. It is essential to proceed rapidly.

- Using the little spatula fill half capsule with the compound.
- In case of a pinned tooth you have to put the compound directly over the pin.

- Insert the right side of capsule over the tooth.
- Remove the cotton roll, previously inserted between gum and cheek, and put it over the fixed capsule.
- Clench teeth for about 5 minutes.
- When the capsule is fixed, remove any excess of paste from around the capsule.

THE FIXING OPERATION DOES NOT ELIMINATE THE GROUNDS THAT CAUSED THE FALL. THIS PRODUCT HAS A STRONG RESISTANCE BUT WE SUGGEST YOU TO CONSULT YOUR DENTIST IF THE PROBLEM PERSISTS. THE QUANTITY OF COMPOUND IS SUFFICIENT TO FIX THREE OR FOUR CAPSULES.

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